

1.2.3

PHYSIOL 1.2.3 with PRS® Preloaded injection system




PHYSIOL 1.2.3 PRELOADED INJECTION SYSTEM FOR 2.2-2.4 MM INCISIONS WITH PRS® TECHNOLOGY

The MicroPure 1.2.3 lens is delivered preloaded in a cartridge, which is simply clipped to the Physiol 1.2.3 single-use injector.

The Physiol 1.2.3 preloaded injection system requires no lens handling which ensures perfect control of asepsis and makes lens injection comfortable and reproducible. Moreover the unique PRS® (Pressure Release System) technology offers an extremely smooth injection in combination with a significant decrease of pressure on the incision.

INJECTION GUIDELINES

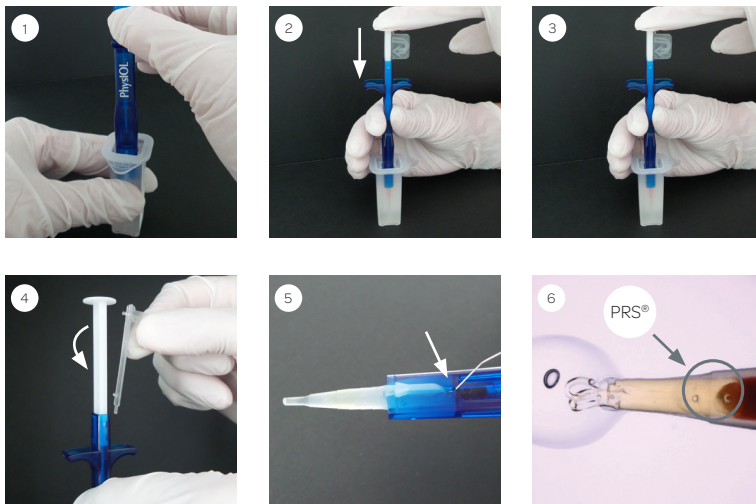

1. Connect the injector vertically onto the preloaded cartridge until you hear the «clip».
2. Push the plunger completely down towards the safety catch and...
3. ...keep the plunger in this position for 3 seconds. This ensures the lens is securely loaded in the cartridge. Then, gently release the plunger.
4. Remove the safety catch by a twist motion.
5. Rinse the IOL with BSS by introducing the cannula of the BSS syringe into the small hole on the body of the injector, and then inject a generous amount of viscoelastic* into the same hole.
6. Push the plunger for injection. When the first two haptics are out of the cartridge, release the plunger a few millimeters to free the two posterior haptics, then push again until the full implantation.

The assembling of the injector onto the cartridge must be performed right before implantation of the IOL.

* Take the viscoelastic solution out of the refrigerator at least one hour before use.

PHYSIOL 1.2.3 WITH PRS® IS AVAILABLE WITH:



Physiol sa/nv - Liège Science Park - Allée des Noisetiers 4 - 4031 Liège - Belgium
t. +32 (0)4 361 05 49 - f. +32 (0)4 361 05 30 - info@physiol.be - www.physiol.eu